

Instructions for Use

Hypodermic Insulin Syringe


100U-Fine Insulin Syringe

Virtually Painless *With Ultrathin needle....*


feather touch....

U100 STERILE SINGLE USE DISPOSABLE INSULIN SYRINGE


- Keep this leaflet. You may need to read it again.
- Consult your health professional before using U100 sterile single use disposal insulin syringe.
- If you have any difficulty using the syringe then ask your doctor or pharmacist.

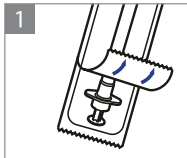


1 Open the 100U-fine insulin syringe box, take a syringe to expose syringe open the wrap as show in image and make sure the insulin is clear and colorless. (Do not use it if it is cloudy or if you see particles; throw it away.)

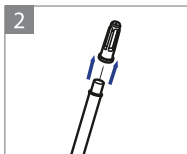

2 To expose needle, pull colored cap straight off being careful not to bend needle

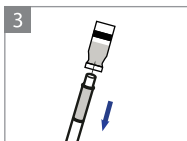


3 Use as directed by the physician. Measure correct dose by lining up the ring nearest the gasket tip with the correct dosage mark on the syringe barrel. To minimize air bubbles, draw you insulin slowly.

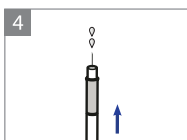

4 **Check for Bubbles:** Before you take the needle out of the vial, check the syringe for air bubbles.


Tap to Release: If bubbles are in the medicine, hold the syringe straight up and tap the side of the syringe until the bubbles float to the top.

Eject the Air: Push the bubbles out with the plunger and draw insulin back in until you have the correct dose.

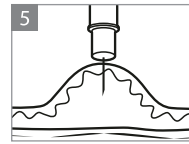

5 **Administer Insulin:** Slowly push in the plunger of the syringe all the way, making sure you have injected all the insulin. Leave the needle in the skin for 10 seconds.

Apply Light Pressure: Pull the needle straight out and gently press on the spot where you injected yourself for several seconds. Don't rub the area.

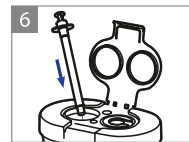

6 **Discard Materials Safely:** After use dispose of the syringe in accordance with local safety standards to prevent misuse or follow your healthcare professional's instructions for throwing away the 100U-fine insuline syringe.



Additional Information:

If you require further information regarding, 100U-Fine Insulin syringe please contact us at info@trionpharma.co.uk



Exclusively distributed by:

TriOn Pharma UK

Cowplain PO8 8DL

TriOn Pharma info@trionpharma.co.uk

EC REP LC & Partners UK Ltd.;
#808, 54 Bloomfield Avenue, Belfast BT5 5AD,
Northern Ireland, UK.

NOTE: This information leaflet is a summary. It may not cover all possible information. If you have questions about this product talk to your doctor, pharmacist or healthcare provider.

DATE OF INFORMATION: 11/2020