

# EasyLube

# STERILE LUBRICATING JELLY

Water-Soluble, Safe & Transparent

## INSTRUCTIONS FOR USE

### COMPOSITION:

EasyLube is a sterile, transparent, water-soluble lubricating jelly with a pH range of 0.5 - 7.0.  
Water, Glycerin, Polyethylene Glycol, Carbomer 140G, Sodium Hydroxide, Methylparaben, Propylparaben.

### INTENDED USE:

EasyLube is a medical device that is a water-soluble, sterile lubricating jelly intended for the smooth insertion of medical devices and examinations, reducing the risk of trauma during procedure, including: Insertion of catheters, Endoscopy, Gynecological examinations, Rectal examinations, Cystoscopy etc.

### INTENDED USER:

The intended users of EasyLube are patients, their carers and /or Healthcare Professionals.

### CONTRAINDICATIONS:

Do not use EasyLube if you are sensitive or allergic to any of the device ingredients.

### WARNING AND PRECAUTIONS:

It is recommended that EasyLube is used in line with local healthcare policies and procedures following a patient assessment by a Healthcare Professional.

Ensure the EasyLube is suitable for the intended use and compatible with other medical devices to be used in conjunction with it. EasyLube is Latex Free. Do not use an EasyLube sachet or tube if it is damaged, or seal is broken. Keep out of reach of children. Apply the required amount of lubrication necessary for the procedure. Not for use in the eye.

Re-use of this medical device may result in infection and/or cross-contamination. Re-sterilisation, reprocessing, cleaning and disinfection may also compromise the product characteristics, resulting in trauma or infection to the patient.

### UNDESIRABLE SIDE EFFECTS:

There are no known side effects associated with using EasyLube, however, in rare cases local irritation or hypersensitivity may occur. Symptoms of irritation or hypersensitivity include redness, itchiness, or blistering. If this occurs, stop using EasyLube and consult your healthcare professional.

### HOW TO USE:

Consult your Healthcare Professional for advice before using EasyLube. Do not use if packaging is damaged or unintentionally opened prior to use.

**SACHETS:** Sterile and single-use. Tear sachet at one end and apply as required.

**TUBES:** Sterile until opened. Remove cap and peel foil from end of tube. Apply as necessary. Close cap after use. Use within 3 months of opening. Discard sooner if the tubes becomes contaminated.

### STERILE DEVICES:

- EasyLub is supplied sterile.
- EasyLube is sterilized by gamma after packaging process.
- Do not re-sterilise.

### STORAGE AND HANDLING:

- Keep dry and out of direct sunlight.
- Reapply cap in between use.
- Unopened packs have a 3-year shelf life.
- Do not use after expiry date.

### DISPOSAL:

EasyLube must be disposed of according to local policies and waste disposal procedures, including any accessories / consumables used with the device.

### PRESENTATION:

Tubes: 42g., 82g.

Sachets:5g.

**UKRP:** Kingsmead Services Limited  
16 Mezzanine Floor, 19-21 Crawford Street, London W1H 1PJ, UK

**EC REP** Kingsmead Services B.V.,  
Zonnehof 36, 2632 BE, Nootdrop, Netherland



TriOn Pharma

Exclusively Distributed by:

**TriOn Pharma Ltd - UK**

Cowplain PO8 8DL, UK

info@trionpharma.co.uk