

SodiEye® 6%

Sodium Chloride 6% Eye Ointment

Preservative Free

For Ocular Use

5g

Instructions for use

SodiEye Eye ointment is a sterile hypertonic ointment containing Sodium Chloride that helps to provide temporary relief from corneal edema.

1. CONTENTS:

SodiEye Eye ointment contains Sodium Chloride 6% w/w. **Other Ingredients:** Liquid Paraffin Heavy, Lanolin Anhydrous, Purified Water, Microcrystalline Wax, White Soft Paraffin.

2. CONTRAINDICATIONS:

Hypersensitivity to Sodium Chloride or to any of the ingredients listed above.

3. INTENDED USE:

SodiEye Eye ointment is a sterile hypertonic ointment containing Sodium Chloride that helps to provide temporary relief from corneal edema caused by surgery, infection, trauma or any other eye conditions.

WARNING: DO NOT USE except under the advice and supervision of a doctor

4. DIRECTION FOR USE:

Apply a small amount of ointment in the eye(s) every 3 to 4 hours, or as directed by a doctor.



Get the SodiEye eye ointment and a mirror.

- Wash your hands.
- Take the tube and twist open the cap to break the tamper evident ring.
- Hold the tube, pointing down, between your thumb and middle finger.
- Tilt your head back. Pull down your eyelid with a clean finger, until there is a 'pocket' between the eyelid and your eye.
- Bring the tube tip close to the eye. Apply a small amount (1/4 inch) or a drop inside the 'pocket' (picture 1). Use

the mirror if it helps.

- Don't touch your eye or eyelid, surrounding areas or other surface with the tube tip. It could contaminate the product left in the tube.
- Gently press on the base of the tube to release one drop of SodiEye eye ointment at a time.
- Don't squeeze the tube, it is designed so that just a gentle press on the bottom is needed. (picture 2)
- If you use the ointment in both eyes, repeat the steps for your other eye.
- Put the cap back on firmly immediately after use.
- White particles may appear on eye lashes post administration of product if the product oozes out of the eyes. Wipe area around eye with clean cloth soaked in lukewarm water after administration of SodiEye if needed.

5. PRECAUTIONS:

- Do not use if allergic to any of the ingredients.
- The tube tip remains sterile till the tube is opened.
- To avoid contamination or possible eye injury, do not touch tube tip to any surface and avoid contact with the eye.
- Close the cap tightly immediately after use.
- Once opened, discard any unused content after 1 month.
- Do not use after expiry date indicated on the carton & tube.
- Do not use if package is damaged or the tamper evident seal is broken.
- Discard the product content if it becomes cloudy or brown.
- Store in a dry, dark & well-ventilated place at a temperature not exceed 25°C.
- Allow 30 minutes between the administration of other eye products.
- Remove contact lenses before use.

6. SIDE EFFECTS:

Common: Eye irritation (Including burning and discomfort), eye pain, eye pruritus and / or visual disturbance.

Blurring of vision immediately after use. Do not operate machinery, use electric tools or drive a car until clear vision is restored.

In the event that you experience side effects not mentioned in this leaflet, or if there is prolonged irritation/pain or severe visual disturbance or any change in your general health, stop use and consult your doctor immediately.

7. INSTRUCTION FOR DISPOSAL:

The device should be disposed as per prevailing regulations where the device is used.

8. HOW SUPPLIED:

SodiEye eye ointment is supplied in 5g pack size in aluminium tube.

Note: This leaflet is specific to SodiEye eye ointment

- KEEP OUT OF SIGHT AND REACH OF CHILDREN
- DO NOT SWALLOW SOLUTION
- FOR EXTERNAL USE ONLY
- NOT FOR INJECTION



| | |
|--|---|
| | Temperature Limitation |
| | Do not use if package is damaged |
| | Do not Re-use |
| | Do not Re-sterilize |
| | Attention, See Instructions for use |
| | Method of sterilization using aseptic processing techniques |
| | Keep away from sunlight |
| | Manufacturing date |
| | Lot number |
| | Should not be used after specification month & date |
| | Authorized representative in the European community |
| | Reference number |



TriOn Pharma

Exclusively Distributed by:
TriOn Pharma Ltd. - UK
Cowplain PO8 8DL, UK
info@trionpharma.co.uk

UKRP: CMC Medical Devices LTD
Office 32 19-21 Crawford Street, London W1H 1PJ - UK



SC Farma Mas Trade SRL STR Valea Seman
NR. 34, Urzici, Jud Praha 16315, Romania

REF 2LER840

Issue No: 01 Date of issue: 11-12-2021